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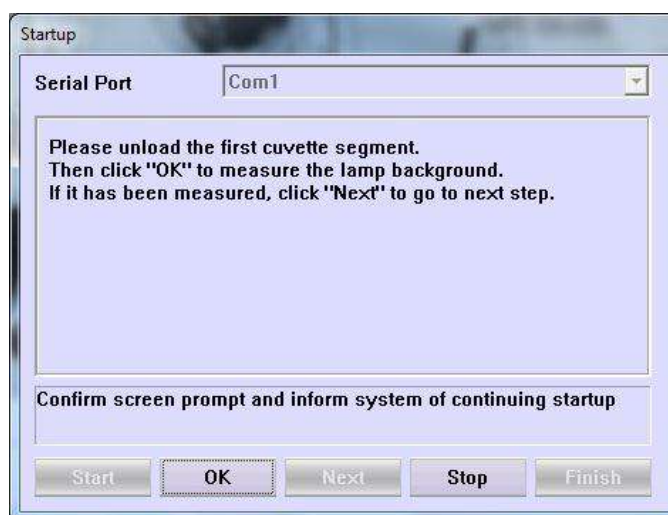
Startup

Serial Port: Com1


Please unload the first cuvette segment.
Then click "OK" to measure the lamp background.
If it has been measured, click "Next" to go to next step.

Confirm screen prompt and inform system of continuing startup

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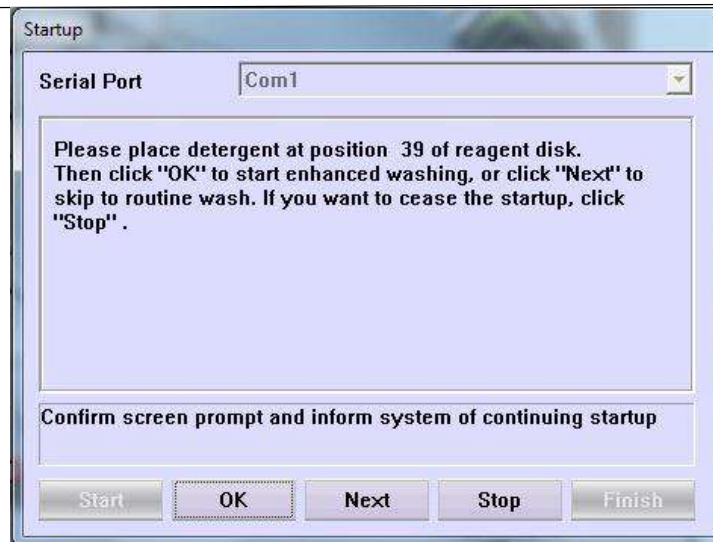
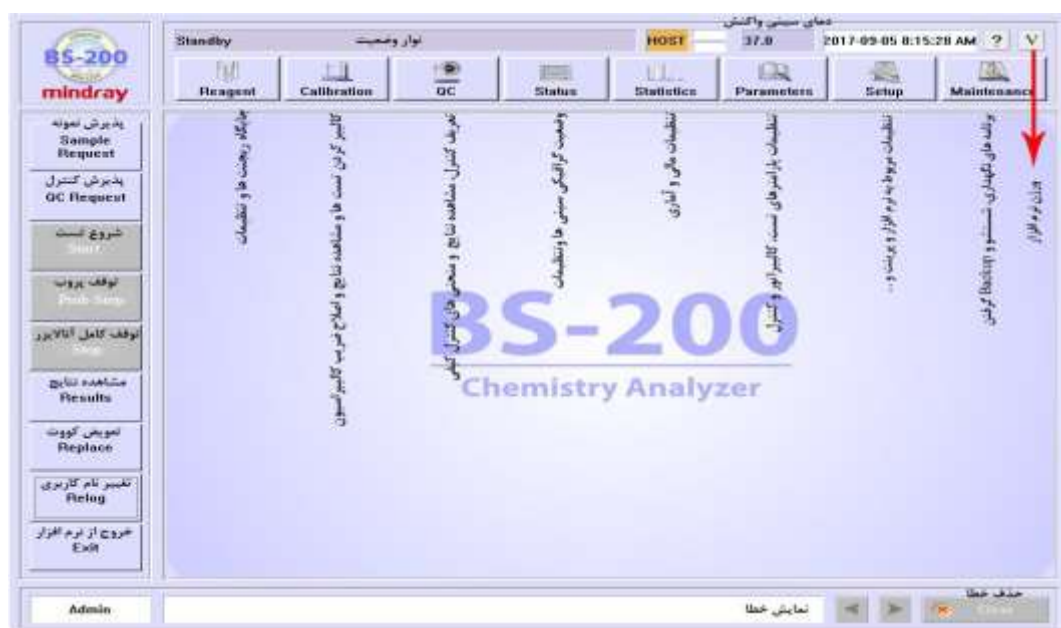
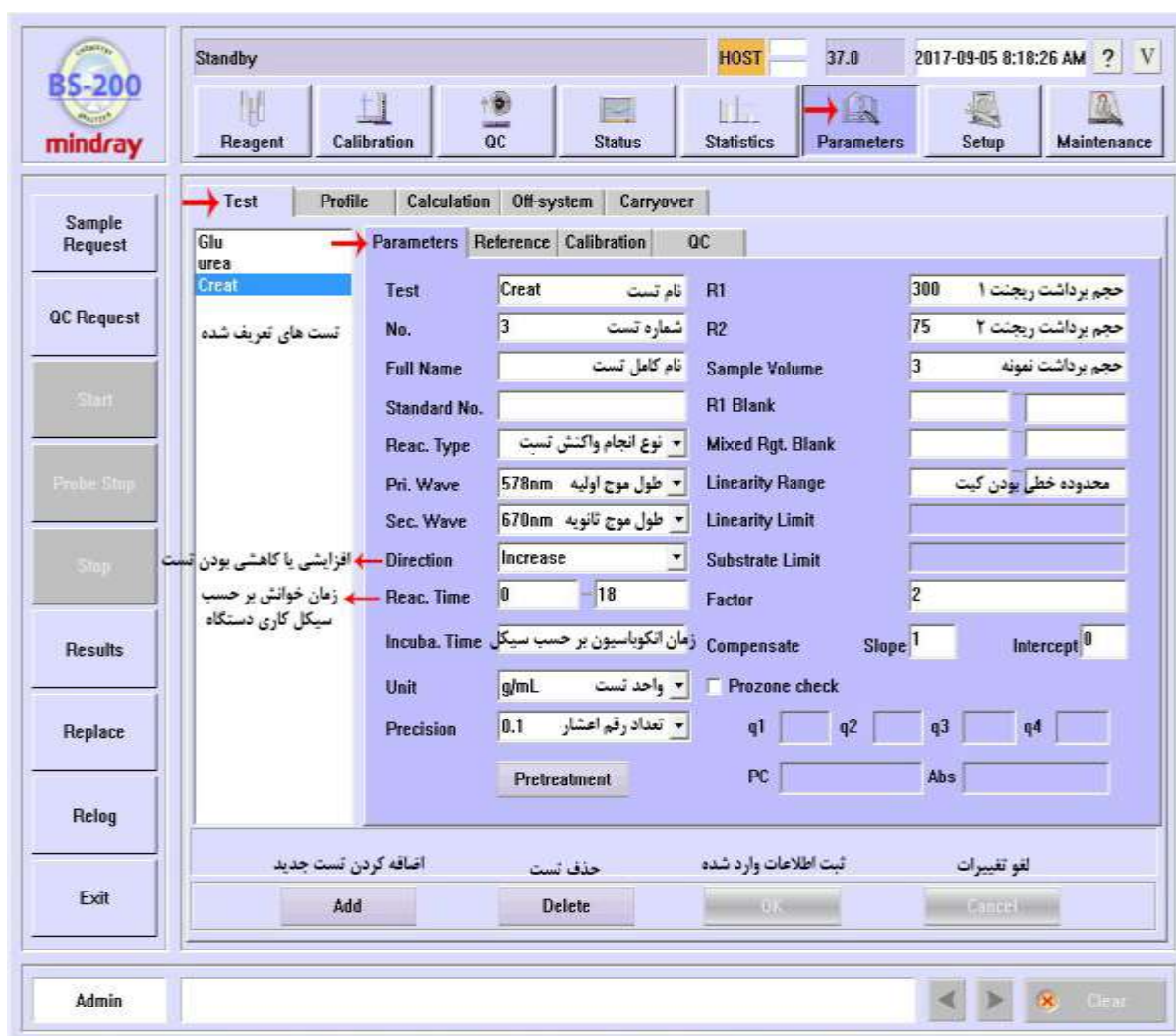
The screenshot shows the BS-200 Mindray software interface. The 'Parameters' tab is selected, and the 'Creat' test is chosen. The interface is divided into several sections:

- Left Panel:** Contains buttons for 'Sample Request', 'QC Request', 'Start', 'Probe Stop', 'Stop', 'Results', 'Replace', 'Re-log', and 'Exit'.
- Top Bar:** Displays 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 8:18:26 AM'.
- Navigation Bar:** Includes buttons for 'Reagent', 'Calibration', 'QC', 'Status', 'Statistics', 'Parameters' (selected), 'Setup', and 'Maintenance'.
- Test Selection:** A list on the left shows 'Glu' and 'urea'. 'Creat' is selected under 'urea'.
- Parameters Table:**

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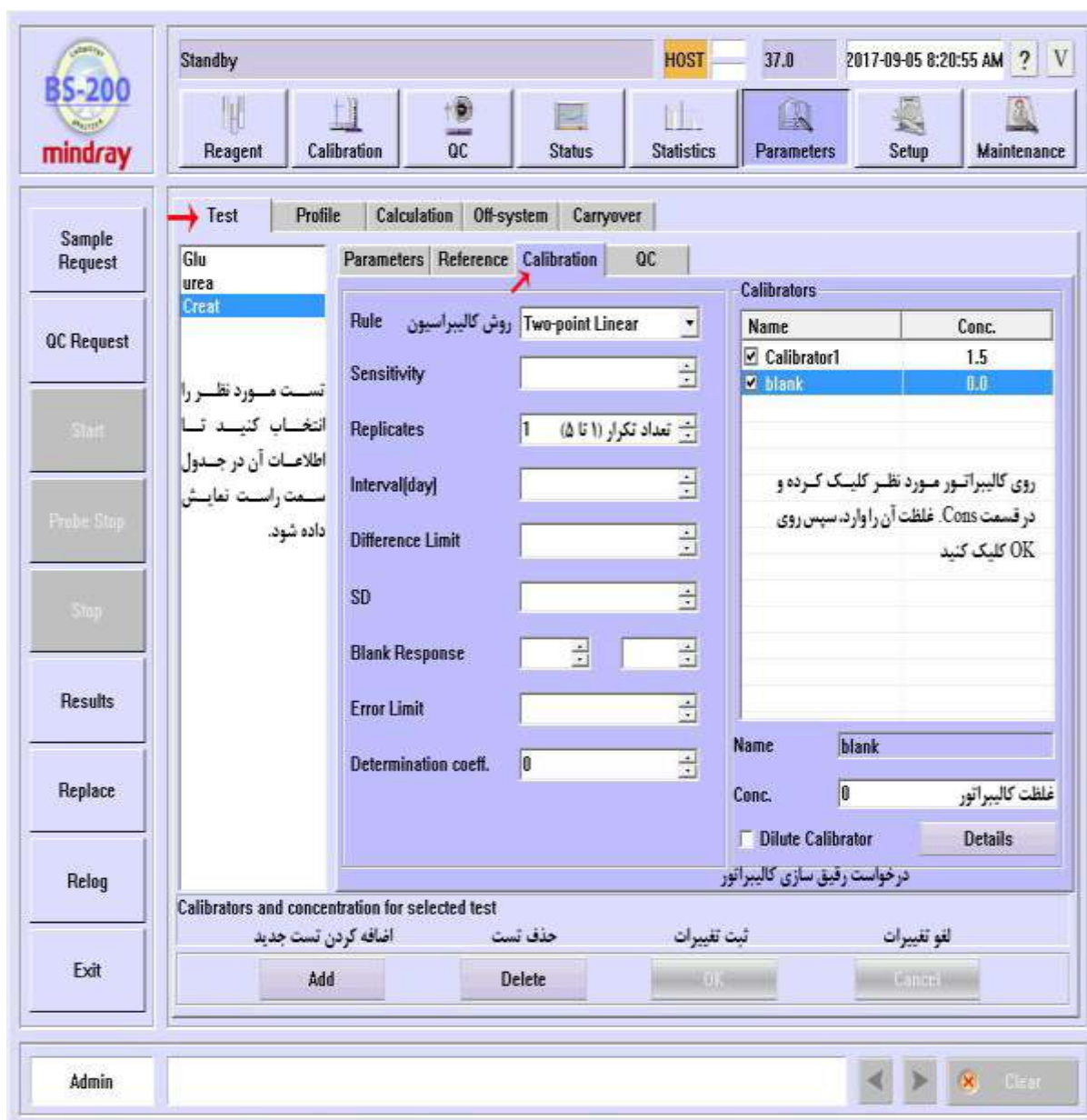Standby HOST 37.0 2017-09-05 8:20:55 AM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Test Profile Calculation Off-system Carryover

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Calibrators and concentration for selected test

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The screenshot shows the BS-200 Mindray software interface. The top bar displays 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 8:21:49 AM'. Below this is a navigation menu with icons for Reagent, Calibration, QC, Status, Statistics, Parameters, Setup, and Maintenance. The 'Parameters' icon is highlighted with a red arrow.

The main window is divided into several sections. On the left is a vertical toolbar with buttons: Sample Request, QC Request, Start, Probe Stop, Stop, Results, Replace, Re-log, and Exit. The top of the main window has tabs: Test, Profile, Calculation, Off-system, and Carryover. The 'Test' tab is active, and within it, the 'Parameters' sub-tab is selected.

The 'Parameters' sub-tab shows a list of tests on the left, including 'Glu', 'urea', and 'Creat'. The 'Creat' test is selected. To the right of the test list, there are checkboxes for 'Westgard Multi-rule' and a list of rules: 1-2S, 1-3S, 2-2S, R-4S, 4-1S, and 10-X. Below these are 'Cum. Sum Check' options: 1.0, 2.7; 1.0, 3.0; and 0.5, 5.1. A 'QC Interval' field is set to 0.

On the right side of the 'Parameters' sub-tab, there is a table with columns 'Name', 'Mean Conc.', and 'SD'. The table contains two rows: 'Control N' with Mean Conc. 1.1 and SD 2.000, and 'Control H' with Mean Conc. 2.0 and SD 2.000. Below the table, there is a text box with the Persian instruction: 'کنترل موردنظر را انتخاب کنید و جدول زیر را کامل کنید' (Select the control you want and complete the table below).

At the bottom of the 'Parameters' sub-tab, there are fields for 'Name' (Control H), 'Mean Conc.' (2), and 'SD' (2). The 'SD' field has a formula $SD = (Max - Min) / 4$ and a label 'انحراف معیار' (Standard Deviation). Below these fields, there is a section titled 'Controls and concentration for selected test' with buttons for 'Add', 'Delete', 'OK', and 'Cancel'.

The bottom of the interface has an 'Admin' section with a text input field and 'Clear' and 'OK' buttons.

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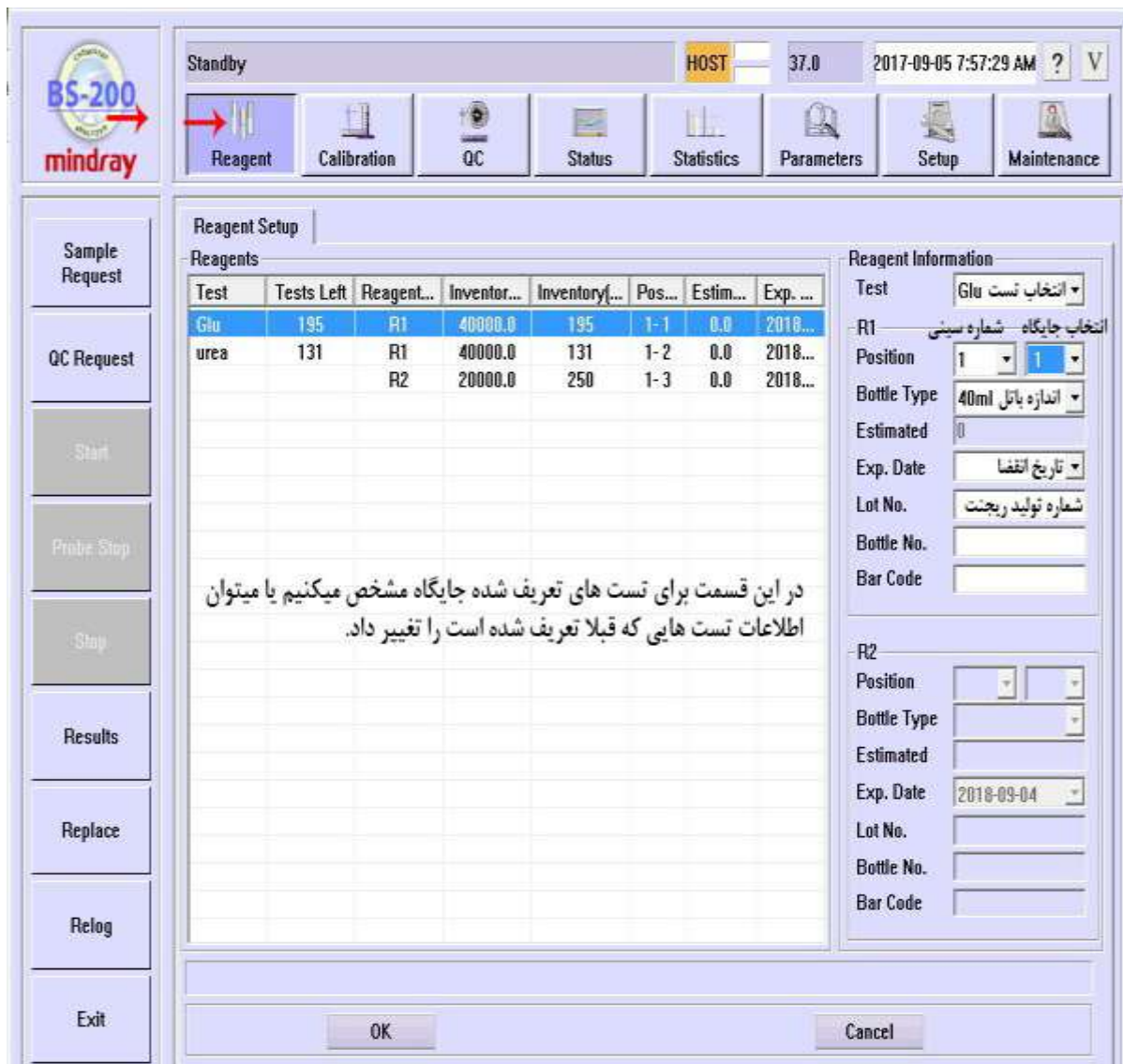
The screenshot shows the 'Reagent Setup' window in the BS-200 Mindray software. The window has a sidebar on the left with buttons: Sample Request, QC Request, Start, Probe Stop, Stop, Results, Replace, Relog, and Exit. The main area is titled 'Reagent Setup' and contains a 'Reagents' table and 'Reagent Information' panels.

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R2
 Position:
 Bottle Type:
 Estimated:
 Exp. Date: 2018-09-04
 Lot No.:
 Bottle No.:
 Bar Code:

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The screenshot shows the BS-200 Mindray software interface. The top bar displays 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 7:52:59 AM'. Below this is a menu bar with icons for Reagent, Calibration (highlighted with a red arrow), QC, Status, Statistics, Parameters, Setup, and Maintenance.

The main window is divided into several sections:

- Left Sidebar:** Contains buttons for 'Sample Request', 'QC Request', 'Start', 'Probe Stop', 'Stop', 'Results', 'Replace', 'Relog', and 'Exit'.
- Top Tabs:** 'Calibration Request' (selected), 'Results', and 'Calibrator'.
- Calibrators Table:**

Name	Lot No.	Position	Exp. Date	Level
Calibrat...	Lot No.	1- 39	2018-09...	Medium
blank	Lot No.	1- 40	2018-09...	Medium
- Tests Table:**

Test	Concentration
Glu	0.0
urea	0
- Calibrator Information:**

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Standby HOST 37.0 2017-09-05 7:55:38 AM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Sample Request QC Request Start Probe Stop Stop Results Replace Relog Exit

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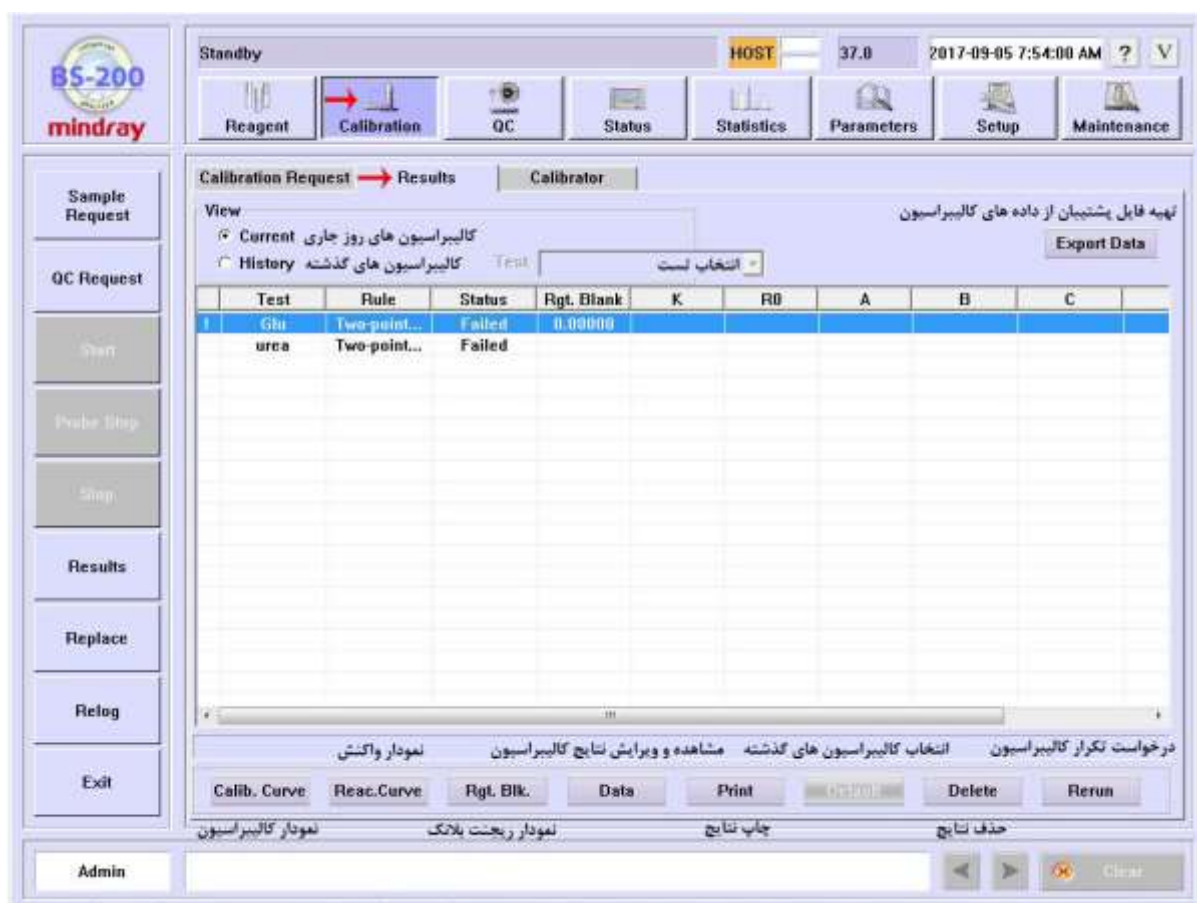
The screenshot shows the BS-200 Mindray calibration software interface. The top bar displays 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 7:54:00 AM'. Below this is a menu bar with icons for Reagent, Calibration (selected), QC, Status, Statistics, Parameters, Setup, and Maintenance. The main window is titled 'Calibration Request' and 'Results'. It has a 'View' section with 'Current' (selected) and 'History' options. The 'Current' view shows a table with columns: Test, Rule, Status, Rgt. Blank, K, RD, A, B, C. The table contains two rows: 'Glu' with 'Two-point...' rule and 'Failed' status, and 'urea' with 'Two-point...' rule and 'Failed' status. The 'History' view is empty. The bottom bar has buttons for 'Calib. Curve', 'Reac. Curve', 'Rgt. Blk.', 'Data', 'Print', 'Delete', and 'Rerun'. The 'Admin' button is at the bottom left.

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Reac. Curve Delete Rerun Change Recalculate OK Cancel Close

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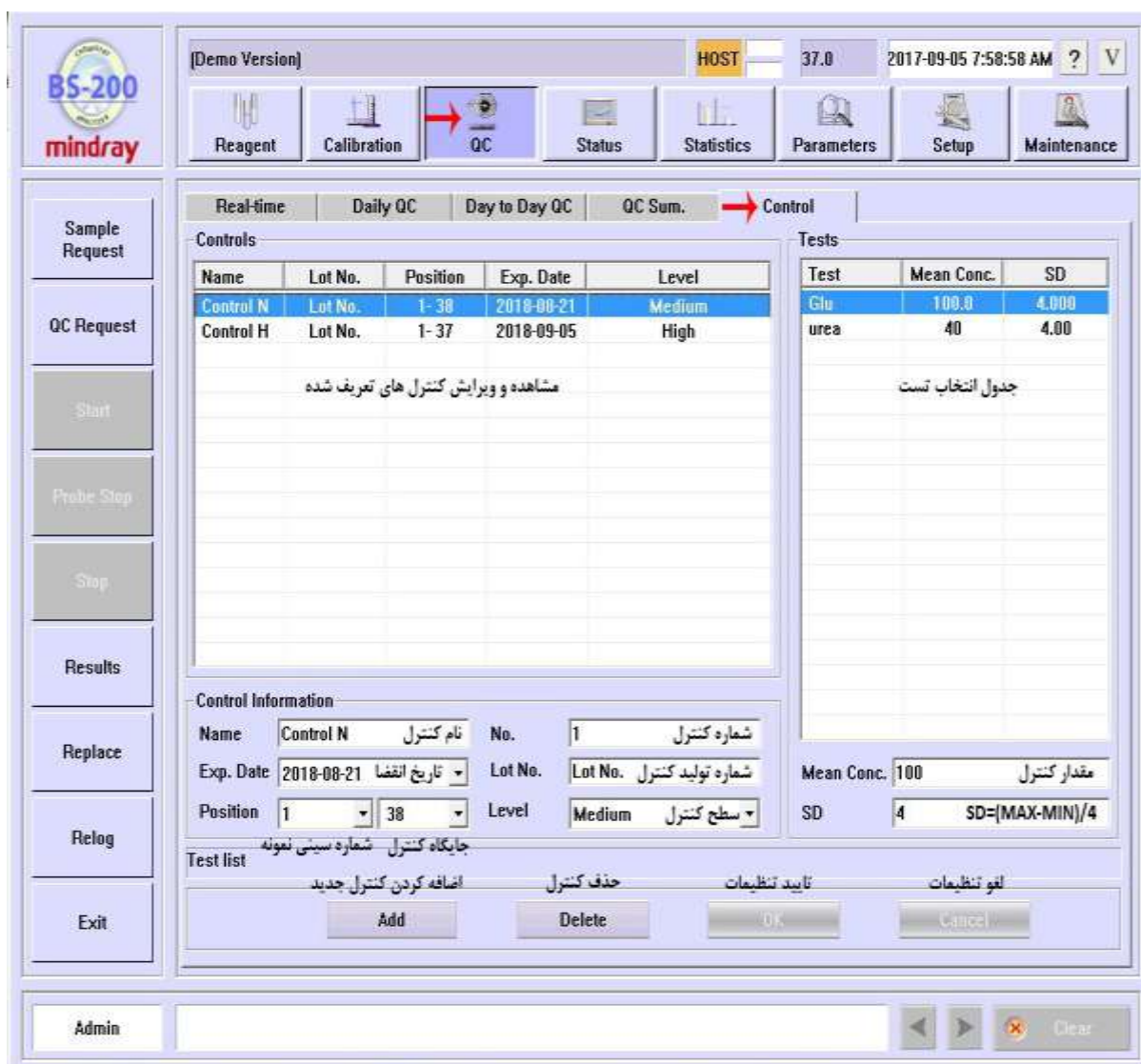
Control

The screenshot shows the BS-200 Mindray software interface. The top bar includes the 'BS-200 mindray' logo, a 'Demo Version' label, and a status bar with 'HOST', '37.0', and the date/time '2017-09-05 7:58:58 AM'. The main menu has icons for Reagent, Calibration, QC (highlighted with a red arrow), Status, Statistics, Parameters, Setup, and Maintenance.

The 'QC' section is active, showing a 'Control' tab. It contains two tables: 'Controls' and 'Tests'.

Name	Lot No.	Position	Exp. Date	Level
Control N	Lot No.	1- 38	2018-08-21	Medium
Control H	Lot No.	1- 37	2018-09-05	High

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Below the 'Tests' table, there is a text box stating: جدول انتخاب تست


The 'Control Information' section includes fields for Name (Control N), No. (1), Exp. Date (2018-08-21), Lot No., Position (1), Level (Medium), and a 'Test list' section with buttons for Add, Delete, OK, and Cancel.

The bottom bar includes an 'Admin' field and a 'Clear' button.

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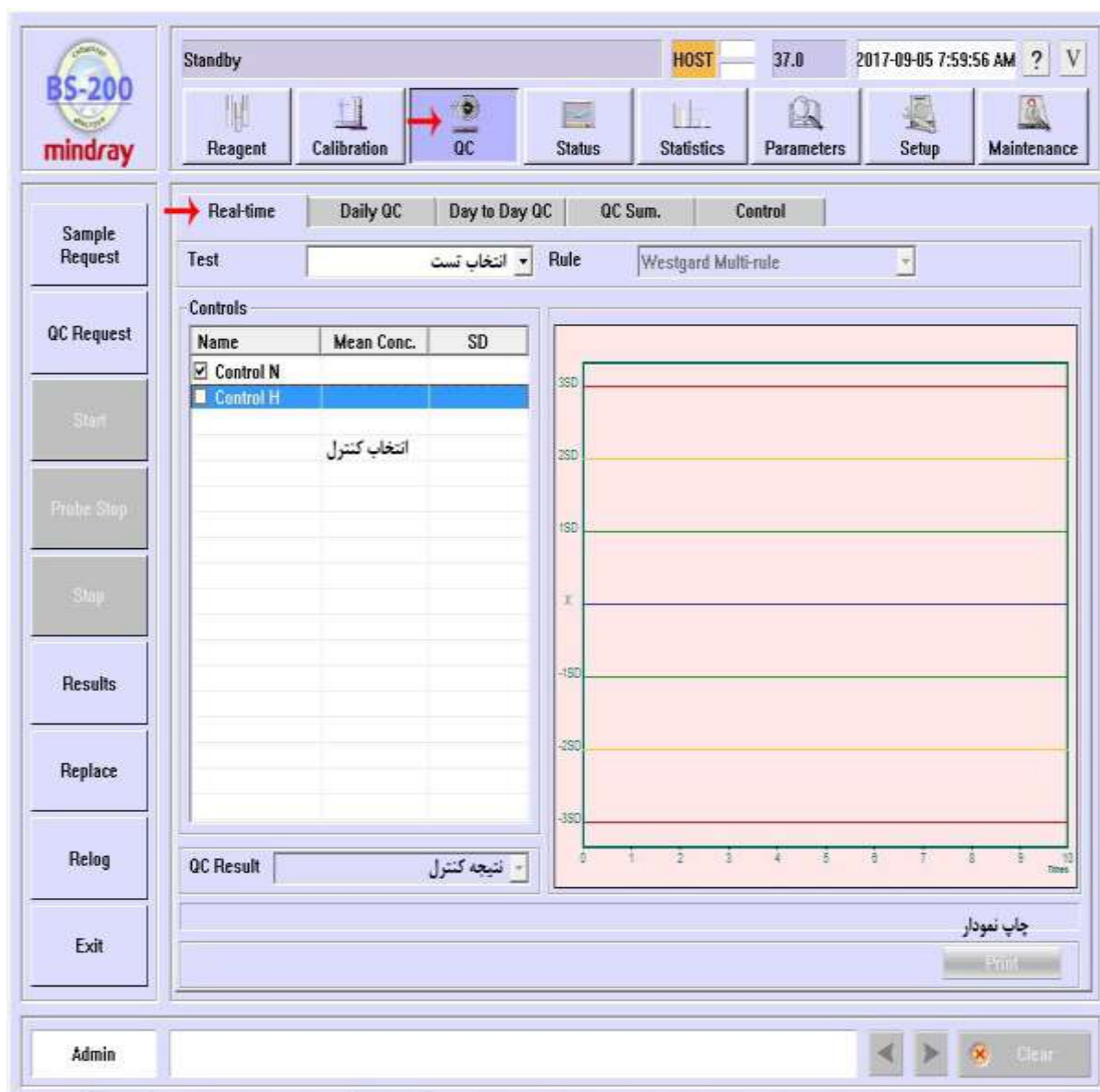Standby HOST 37.0 2017-09-05 7:59:56 AM ? V

Reagent Calibration **QC** Status Statistics Parameters Setup Maintenance

Real-time Daily QC Day to Day QC QC Sum. Control

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
The screenshot shows the BS-200 Mindray software interface. The top bar includes 'Standby', 'HOST', '37.0', and the date/time '2017-09-05 8:00:32 AM'. Below this is a navigation bar with buttons: Reagent, Calibration, QC (highlighted with a red arrow), Status, Statistics, Parameters, Setup, and Maintenance. The main window has a left sidebar with buttons: Sample Request, QC Request, Start, Print Stop, Stop, Results, Replace, Rlog, and Exit. The main area is divided into tabs: Real-time, Daily QC (selected), Day to Day QC, QC Sum., and Control. Under the Daily QC tab, there are options for 'مشاهده به صورت جدول' (Tabular Data) and 'مشاهده به صورت نمودار' (Graphical Data). A 'Test' dropdown is set to 'انتخاب تست', and a 'Date' dropdown is set to '2017-09-05'. The 'Rule' is set to 'Westgard Multi-rule'. Below these are 'Control 1' and 'Control 2' dropdowns, both set to 'انتخاب کنترل'. A table with columns 'Test', 'Control', 'Mean Conc.', 'SD', 'Unit', 'Actual Result', and 'Request Date' is shown. At the bottom, there are buttons for 'Refresh', 'چاپ نتایج' (Print Results), 'حذف نتایج روز جاری' (Delete Today's Results), 'مشاهده نمودار واکشی' (View Leaking Diagram), and 'ارسال نتایج به LIS' (Send Results to LIS). An 'Admin' field is at the bottom left.

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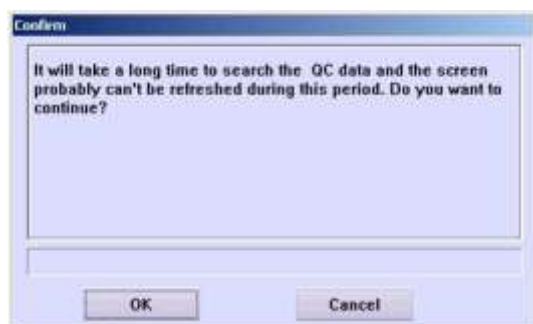The screenshot shows a 'Confirm' dialog box with the text: 'It will take a long time to search the QC data and the screen probably can't be refreshed during this period. Do you want to continue?'. There are 'OK' and 'Cancel' buttons at the bottom.

The screenshot displays the BS-200 mindray software interface. At the top, there's a status bar with "Standby", "HOST", "37.8", and the date/time "2017-09-05 8:01:47 AM". Below this is a navigation bar with icons for Reagent, Calibration, QC (highlighted), Status, Statistics, Parameters, Setup, and Maintenance.

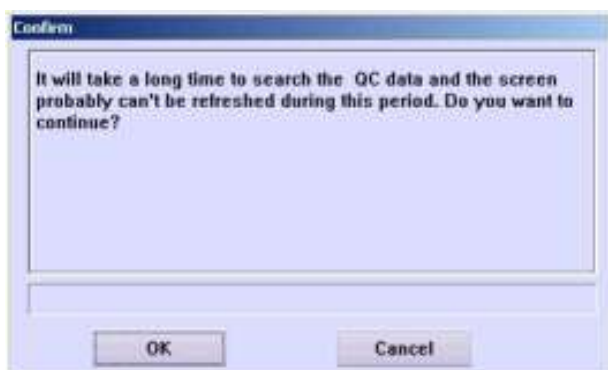
The main window has a left sidebar with buttons: Sample Request, QC Request, Start, Probe Stop, Stop, Results, Replace, Relog, and Exit. The central area shows the "QC" section with tabs for Real-time, Daily QC, Day to Day QC (selected), QC Sum., and Control.

In the "Day to Day QC" tab, there are options for "مشاهده به صورت جدول" (Tabular Data) and "Graphical Data". Under "Tabular Data", there are fields for Test (انتخاب تست), Rule (Westgard Multi-rule), Date (2017-09-05), and Control 1 & 2 (انتخاب کنترل 1 و 2).

A large table below these fields lists test results with columns: Test, Control, Mean Conc., SD, Unit, Actual Result, and Request Date. The table is currently empty.


At the bottom, there are buttons for Refresh, Print, Delete, Reac Curve, and Send. A footer bar includes an Admin field and navigation arrows.

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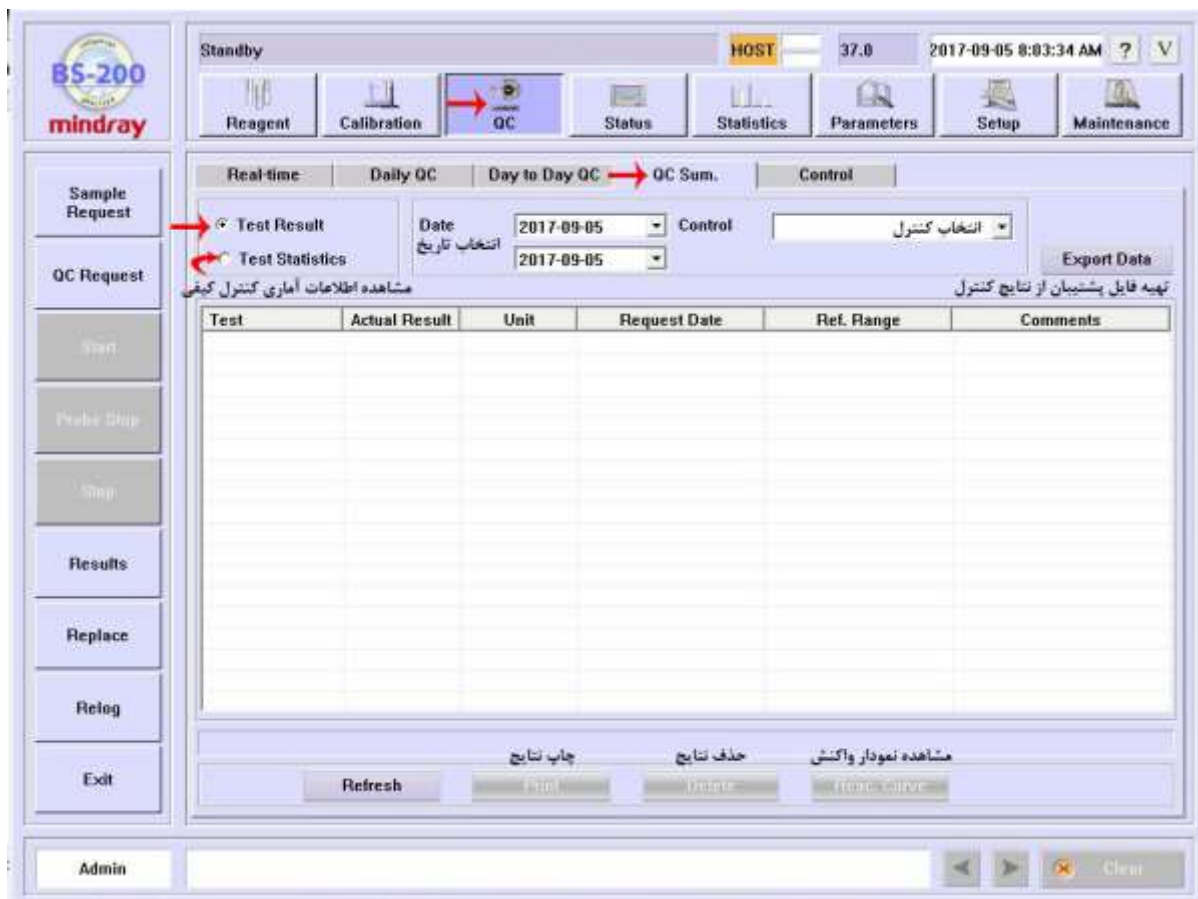
The screenshot shows the BS-200 Mindray software interface. The top menu bar includes options like Reagent, Calibration, QC, Status, Statistics, Parameters, Setup, and Maintenance. The QC Sum screen is active, showing a table for Test, Actual Result, Unit, Request Date, Ref. Range, and Comments. The table is currently empty. The interface also includes buttons for Refresh, Print, Delete, and Home Curve. The date and time are displayed as 2017-09-05 8:03:34 AM.

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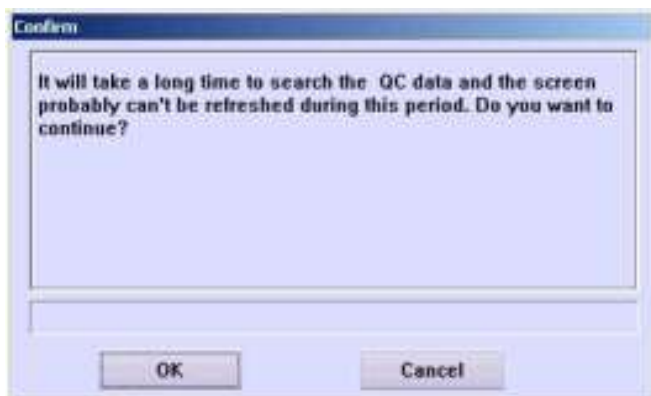Confirm

It will take a long time to search the QC data and the screen probably can't be refreshed during this period. Do you want to continue?

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The screenshot shows the 'Sample Request' window of the BS-200 Mindray software. The interface is in Persian. Key elements include:


- Sample Information:** Fields for 'No.' (7), 'Position' (7), and 'Patient' (نام بیمار).
- Tests:** A list of tests including GLU, Creat, and Urea.
- Table:** A table with columns 'Sa...', 'Po...', and 'Tests' showing sample positions and test results.
- Buttons:** 'Sample Request' (highlighted with a red arrow), 'Change Pos.' (highlighted with a red arrow), 'Print List', and 'Download'.
- Labels:** 'جدول انتخاب تست' (Test Selection Table) and 'جدول نمونه های پذیرش شده' (Accepted Sample Table).

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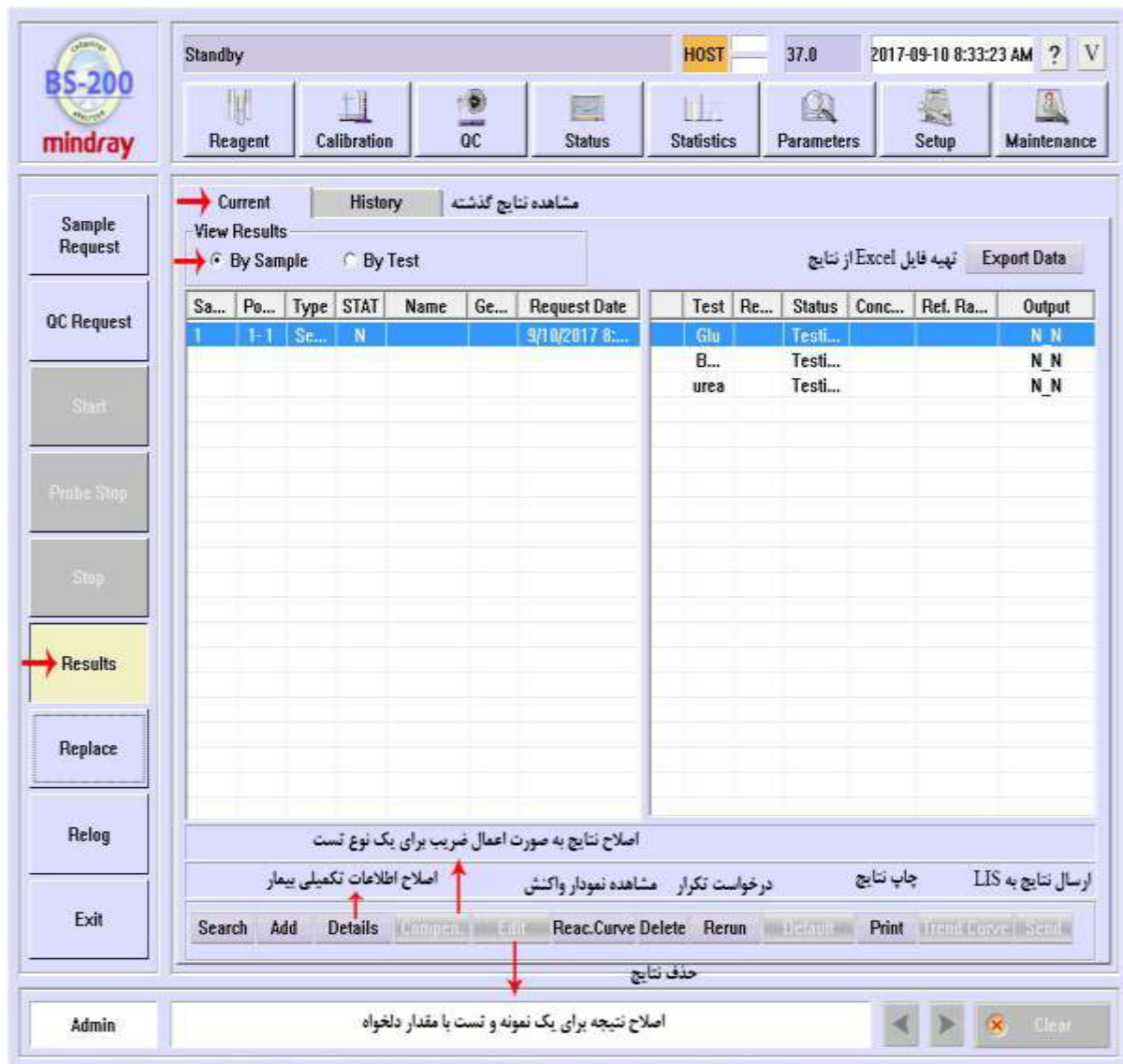Search Add Details Unopen Edit Reac.Curve Delete Rerun Reprint Print Trend Curve Send

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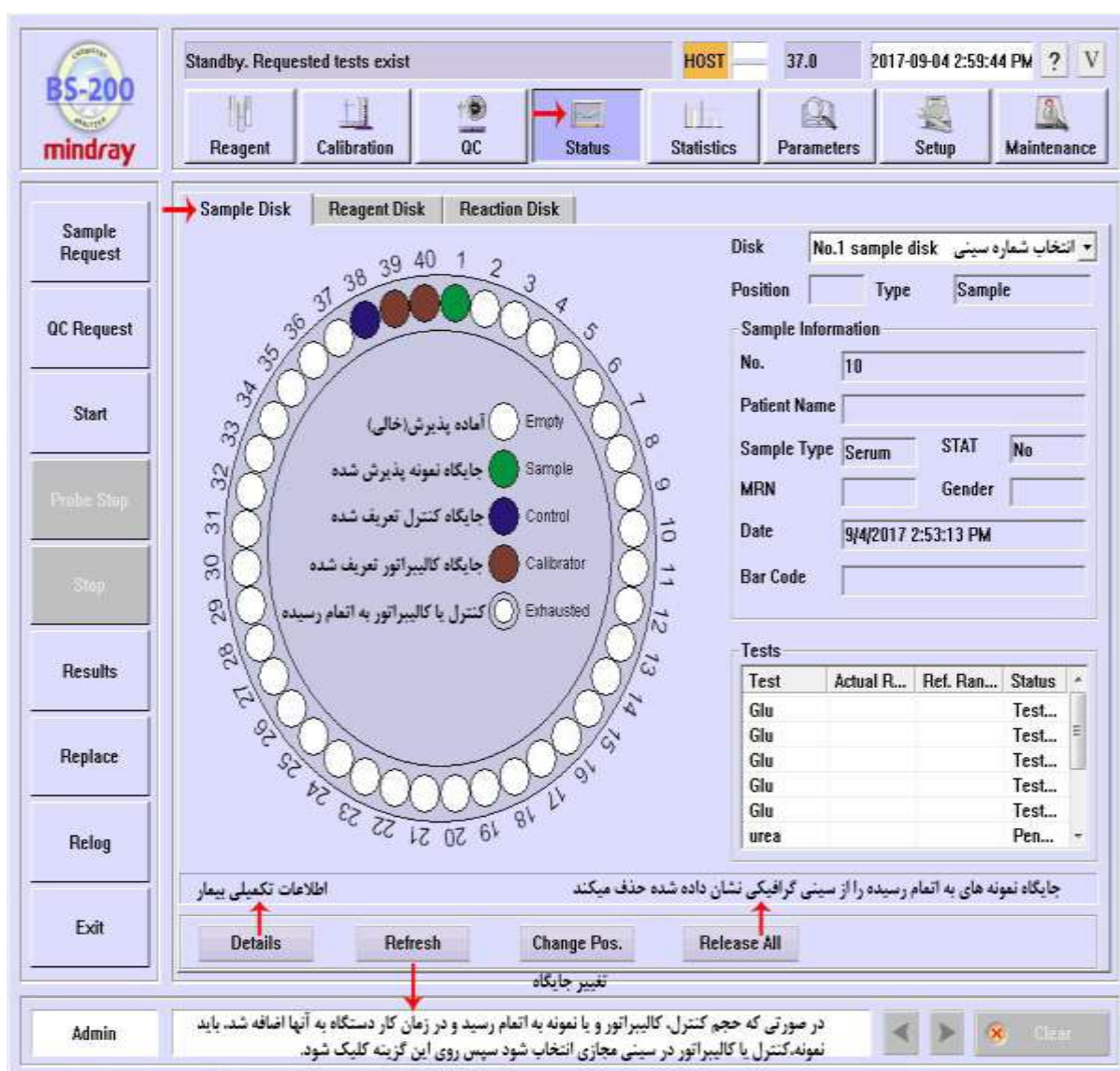
HOST 37.0 2017-09-04 2:59:44 PM ? V

Reagent Calibration QC Status Statistics Parameters Setup Maintenance

Sample Disk Reagent Disk Reaction Disk

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Details Refresh Change Pos. Release All

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
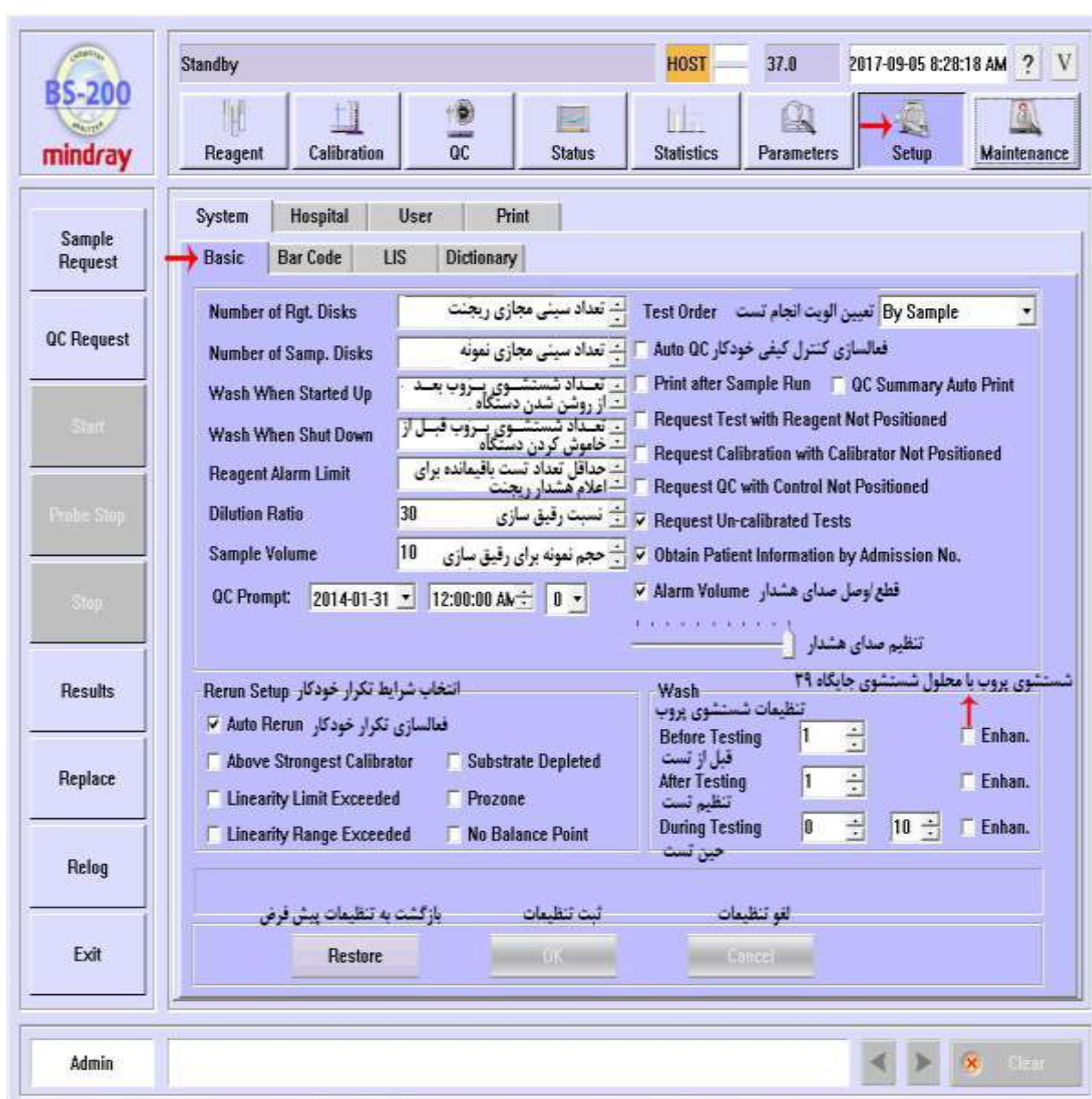
The screenshot shows the 'Setup' window of the BS-200E Mindray software. The window is divided into several sections:

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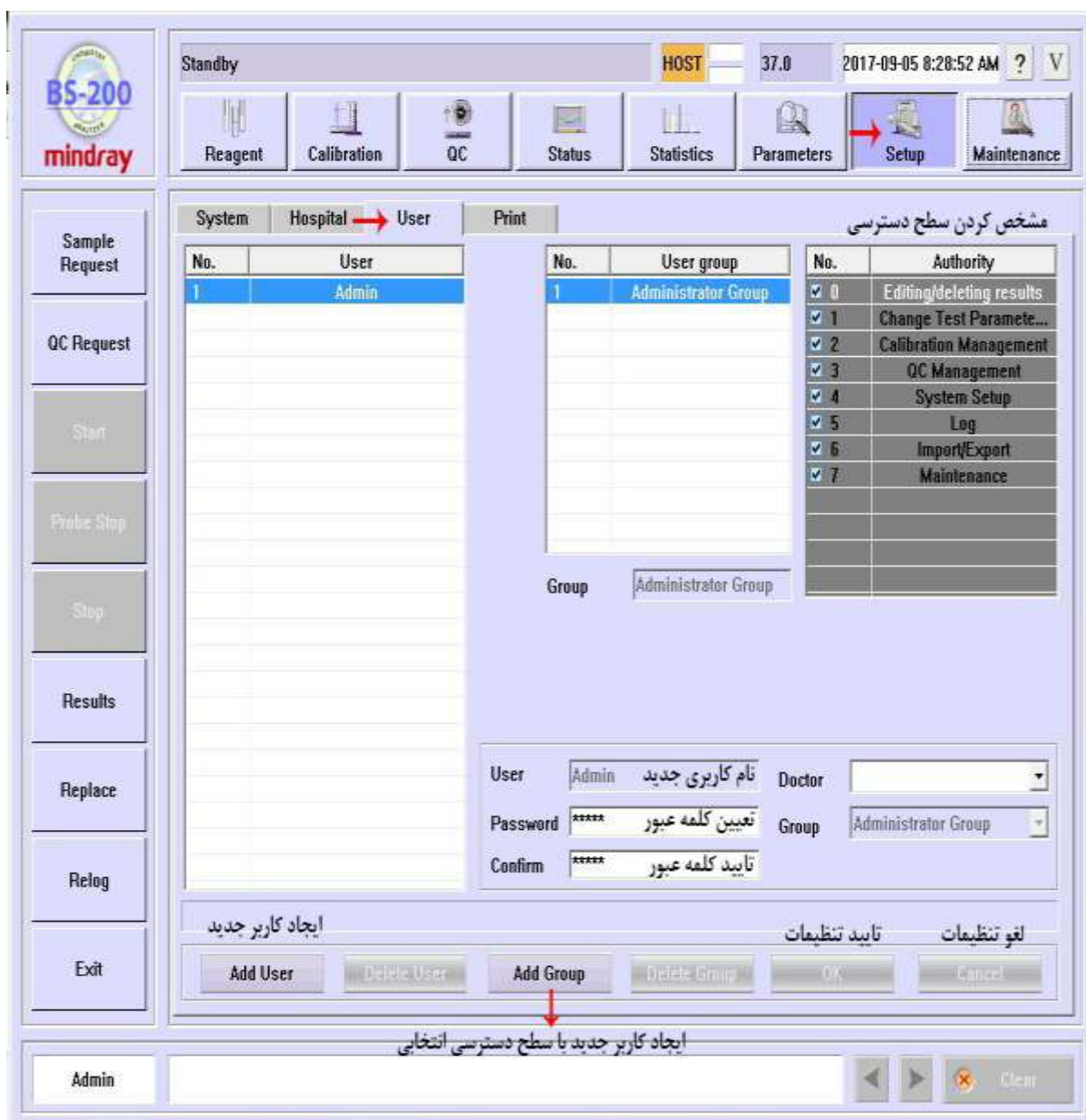The screenshot shows the BS-200 Mindray software interface. The top bar includes the 'BS-200 mindray' logo, a 'Standby' status indicator, a 'HOST' field with the value '37.0', and a timestamp '2017-09-05 8:28:52 AM'. Below this is a navigation bar with buttons for 'Reagent', 'Calibration', 'QC', 'Status', 'Statistics', 'Parameters', 'Setup', and 'Maintenance'. The 'Setup' button is highlighted with a red arrow.

The main window is divided into several sections. On the left is a vertical menu with buttons for 'Sample Request', 'QC Request', 'Start', 'Probe Stop', 'Stop', 'Results', 'Replace', 'Releg', and 'Exit'. The central area has tabs for 'System', 'Hospital', and 'User', with the 'User' tab selected. Below the tabs are three tables:

No.	User
1	Admin

No.	User group
1	Administrator Group

No.	Authority
☒ 0	Editing/deleting results
☒ 1	Change Test Paramete...
☒ 2	Calibration Management
☒ 3	QC Management
☒ 4	System Setup
☒ 5	Log
☒ 6	Import/Export
☒ 7	Maintenance

Below the tables is a 'Group' dropdown menu set to 'Administrator Group'. At the bottom of the central area is a form for adding a new user or group:

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At the bottom of the interface, there are buttons for 'Add User', 'Delete User', 'Add Group', and 'Delete Group'. The 'Add Group' button is highlighted with a red arrow. Below these buttons are buttons for 'OK' and 'Cancel'. At the very bottom, there is a status bar with the text 'ایجاد کاربر جدید با سطح دسترسی انتخابی' and a 'Clear' button.

BS200E

:Maintenance

:Daily Maint

The screenshot shows the BS-200 Mindray software interface. The top bar includes the 'Standby' status, 'HOST' label, '37.0' temperature, and the date '2017-09-05 8:30:20 AM'. The main menu has tabs for Reagent, Calibration, QC, Status, Statistics, Parameters, Setup, and Maintenance. The 'Maintenance' tab is active, showing a 'Daily Maint' section with a table of system parameters. The table has columns for 'System Status', 'Dark Current', and 'Background'. The 'Light Intensity' parameter is highlighted with a red arrow and a value of 68%. Below the table, there are buttons for 'Startup Check', 'Wash', 'Enhanced', 'Dark Current', and 'Reset'. Red arrows point to the 'Startup Check' button and the 'Light Intensity' parameter. The bottom status bar shows 'Failure' and 'Tested by 39'.

System Status	Dark Current	Background
Reaction Temp. 37.00	340	44765
Preheat Temp.	405	44765
Waste Tank	450	44765
Deionized Water	510	44765
Printer Idle	546	44765
Light Intensity 68%	578	44765
Unit Status	630	44765
Main Unit	670	44765
Reaction Unit	Reference	44765
Sample Unit		
Reagent Unit		
Temperature Unit		
Mixing Unit		

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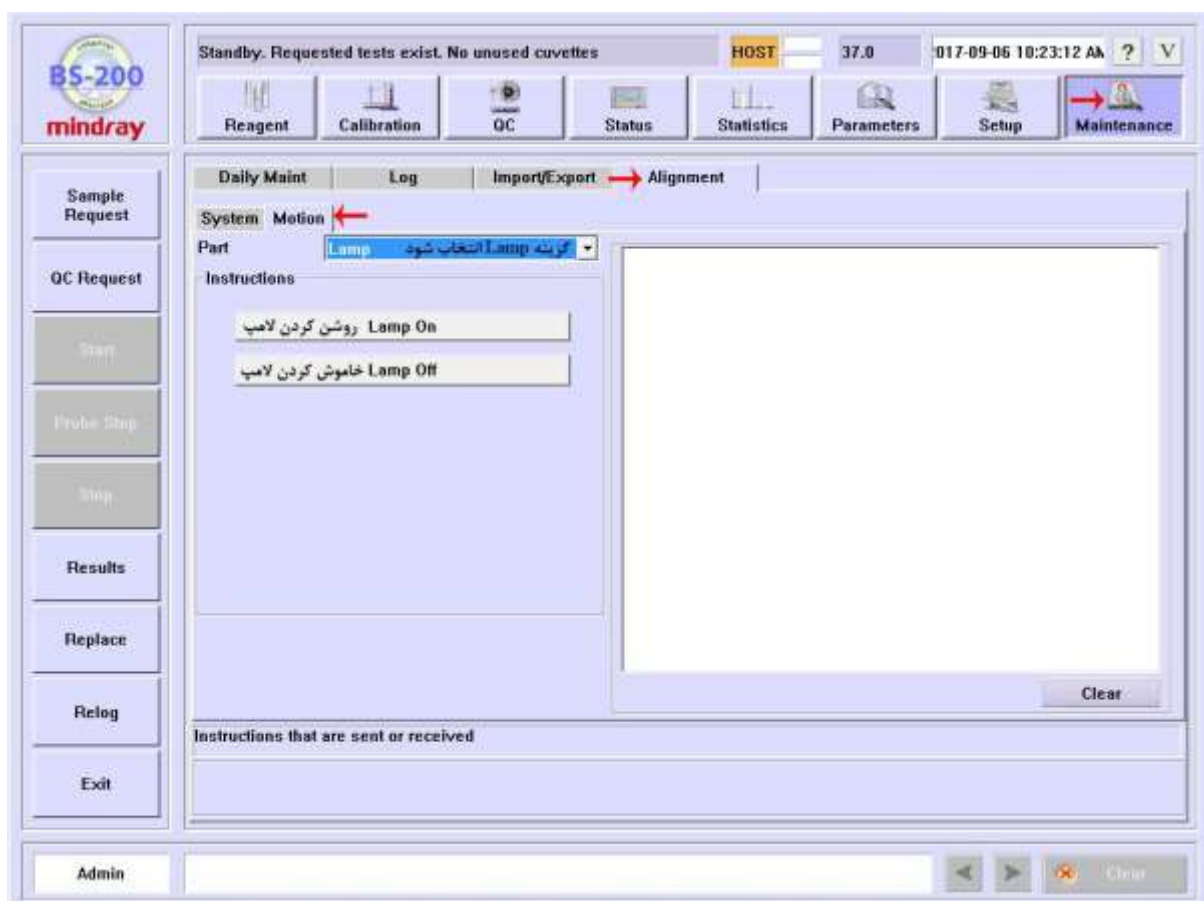
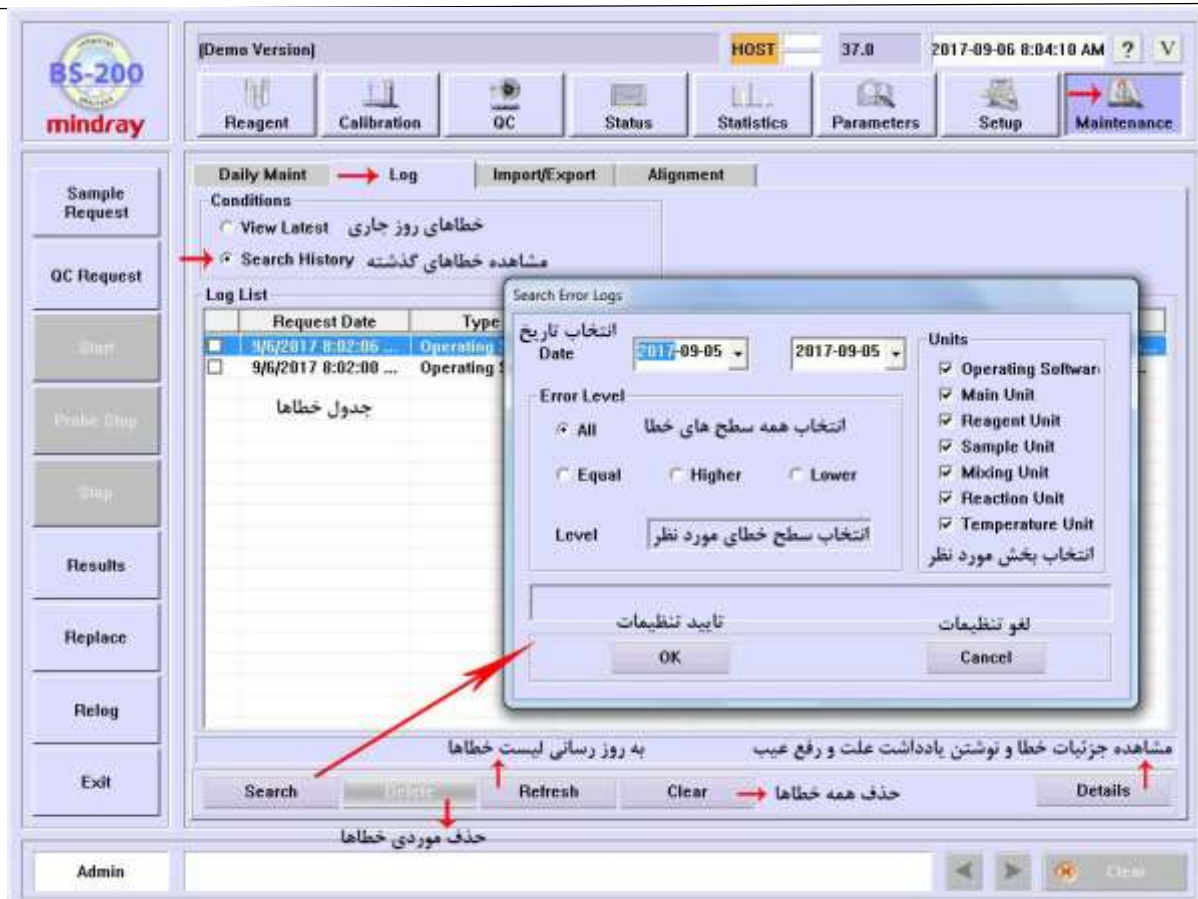
The screenshot shows the BS-200 Mindray software interface. The top status bar displays "Standby. Requested tests exist. No unused cuvettes", "HOST", "37.0", and the date/time "2017-09-06 9:56:17 AM". The main menu includes "Reagent", "Calibration", "QC", "Status", "Statistics", "Parameters", "Setup", and "Maintenance". The "Import/Export" option is highlighted in red. Below the menu, the "Import/Export" tab is active, showing options for "Calibration", "QC", "Sample Analysis", and "System". The "Test Information" section on the left lists various test parameters. The "Tests" section on the right shows a table with columns for "Glu", "urea", "Creat", and "Test6". The table contains data for "BUN" and "Calculation2". A red text overlay in the table area reads: "توجه: به بازه های تاریخی در هنگام تهیه فایل پشتیبان دقت شود". At the bottom, there are buttons for "Test Update", "Import", "Export", and "None". Red arrows point from the "Import" and "Export" buttons to the text "تهیه فایل پشتیبان از داده های انتخابی" (Prepare backup file from selected data). The "Admin" button is located at the bottom left.

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